



Medical Device 0481
Hemorrhoid Gel

- **Hygienic treatment of external hemorrhoids**
- **Relief of hemorrhoid symptoms such as burning, itching and pain caused by swelling hemorrhoids**
- **Relief of local irritation and external discomfort**

General recommendations:

Carefully read the instructions for use prior to using UnoPresto® Hemorrhoid Gel, even if you have used this medical device before. This information may have been amended to take account of new research and findings in the time since you opened your last package. This medical device can be obtained without prescription. In spite of this, take care when using UnoPresto® Hemorrhoid Gel in order to obtain a good result. Keep these instructions for use in case you need to consult them again.

Application:

UnoPresto® Hemorrhoid Gel is a medical device for the treatment of external hemorrhoids and other minor anorectal conditions such as anal itching and fissures. This protective gel reduces irritation, swelling, itching, pain and burning, and prevents contact of the irritated skin with aggravating liquid or stool from the rectum.

Mechanism of action:

UnoPresto® Hemorrhoid Gel forms a protective layer with excellent moisturizing and anti-inflammatory properties. The layer protects the area around the anus and restores the structural integrity of veins and normal blood flow to the area around the anus. It reduces inflammation and the risk of infection, not because it kills bacteria but because it prevents them from penetrating the skin. It helps to resolve a prolapsed Hemorrhoid before it grows large and produces a smoothing effect for the passage of hard stools.

Dosage and use (for adults):

- Squeeze the tube of UnoPresto® Hemorrhoid Gel and apply a small amount on your finger.
- Apply the gel around the outside of your anus with your finger.
- Wash your hands thoroughly with warm water and soap after each application.
- Apply the gel to the affected area two to four times a day, preferably after a bowel movement. Do this for 7-14 days.

Composition:

Aqua, Glycerin, Betaine, Panthenol, Ammonium Acryloyldimethyltaurate/VP Copolymer, Laureth-9, Phenoxyethanol, Xanthan Gum, Allantoin, Ethylhexylglycerin, Sodium Carboxymethyl Betaglukan, Opuntia Ficus-Indica Stem Extract, Sodium Hyaluronate.

Possible side effects:

Irritation could be an adverse effect.

Use during pregnancy and lactation:


Hemorrhoid Gel spray is not recommended during pregnancy and lactation due to a lack of adequate data.



Warnings:

- Keep out of reach of young children;
- If you are suffering from hypersensitivity reactions, stop using Hemorrhoid Gel;
- Check the list of ingredients to know if you are allergic to any of the ingredients;
- Do not use after the expiry date stated on the packaging.

Content: 40 ml OR 20 ml

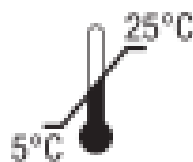

Storage instructions:

Store at a temperature of 5-25°C in a dry and dark place in the original container and out of reach of young children. Do not use after the expiry date stated on the packaging.



Final date of use:

We advise against using Hemorrhoid Gel spray after the expiry date, which is stated on the packaging after 'expiry date'. Hemorrhoid Gel spray can be kept for 1 month after opening.

Medical device:

UnoPresto® Hemorrhoid Gel is a class IIa in accordance with the Medical Device Directive 2007/47/EC.

Date on which these instructions for use were last revised: June 2020 – 080291BY-V2.2



CORNUS GmbH,
Zentgrafenstraße 128,
34130 Kassel, Deutschland

CE 0481